


Tumble Forms²
With Anti-Microbial Protection

Junior Carrie® Potty Seat

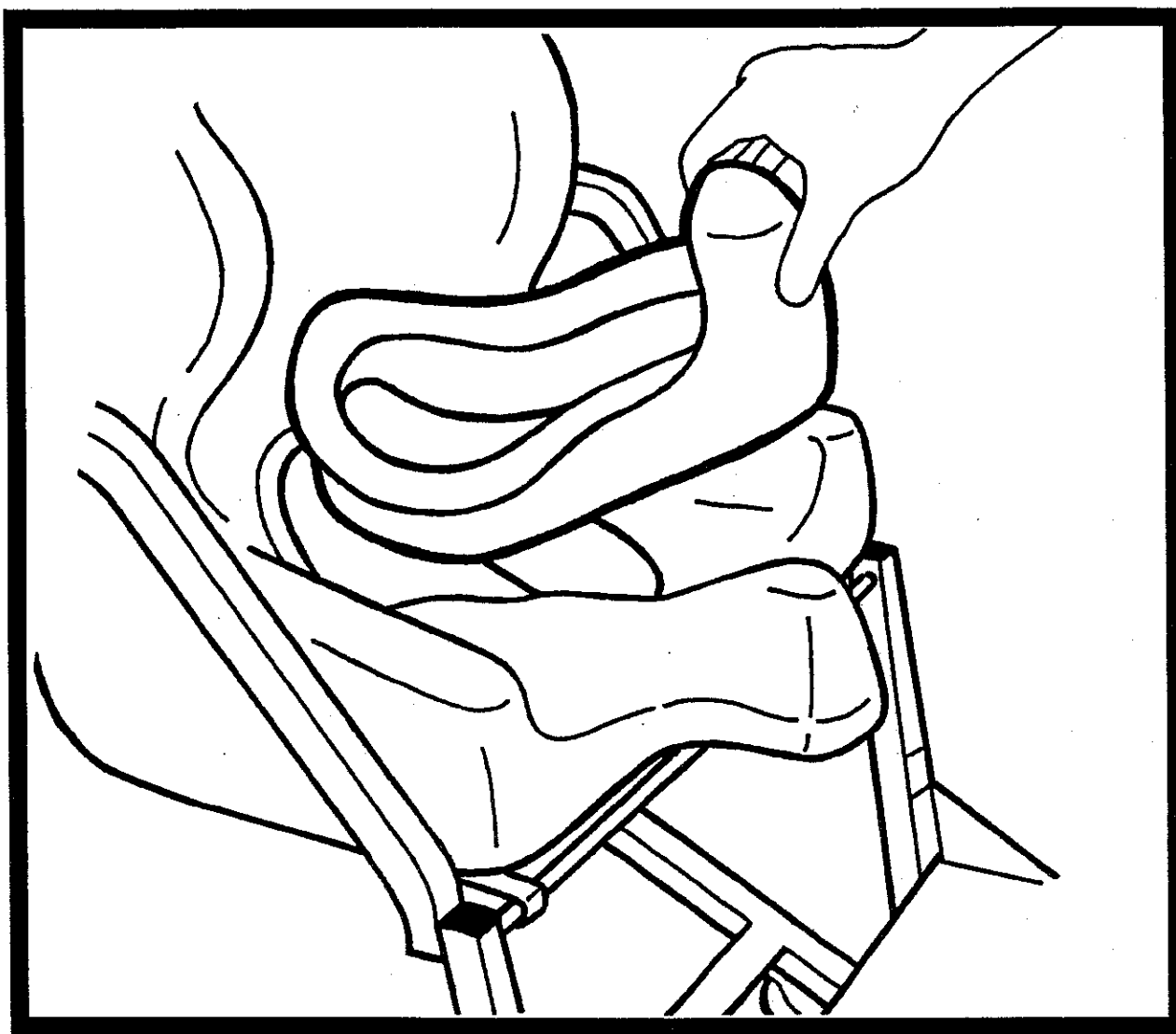


Instruction Manual


Please Note: Sammons Preston does not accept returns on hygienic products that have been used. The Carrie Potty Seat has been packaged so that a child can be harnessed into the seat for size verification without removing the plastic covering. Once the covering has been removed, the seat will be deemed to have been used and will not be returnable except for warranty claims covered under the one year warranty period. Tumble Forms, Inc. Will not inspect any Carrie Potty seat returned for warranty claims that has not been thoroughly cleaned and disinfected.

The Carrie Potty Seat is designed to position children with mild to moderate neuromuscular problems for toileting. It is shaped to keep the legs slightly abducted and flexed to aid in the toileting process.



The Carrie Potty Seat can be used independently as a commode, or over a standard toilet by simply removing the commode pail. Please note that the potty seat may not fit over some toilets. Accessory kit number 2771RJ raises the Junior Potty Seat 3" to fit over handicap toilets. The assembly instructions for the Junior Potty Riser may be found on page 7 of this manual. The seat has a number of adjustable features including the footrest height, the tray depth, and the seat orientation.

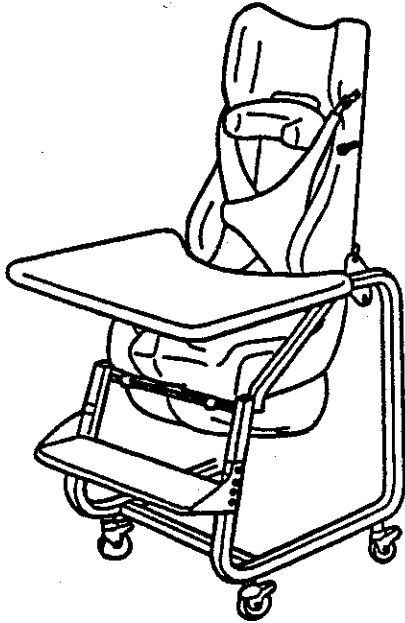


To assemble the potty seat, place the seat into the base frame, sliding the pivot hooks around the front crossbar as shown below.

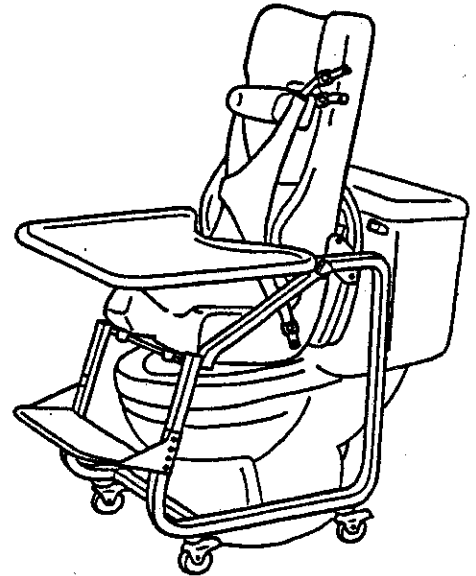

Next, press in the seat adjustment buttons and rotate the back until the buttons pop out into the appropriate adjustment holes. The seat has an upright and a reclined seating position.


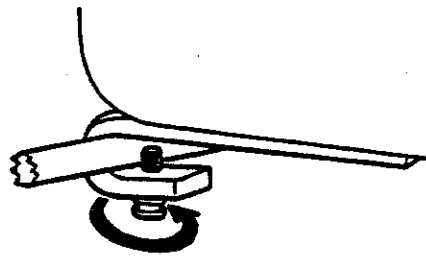
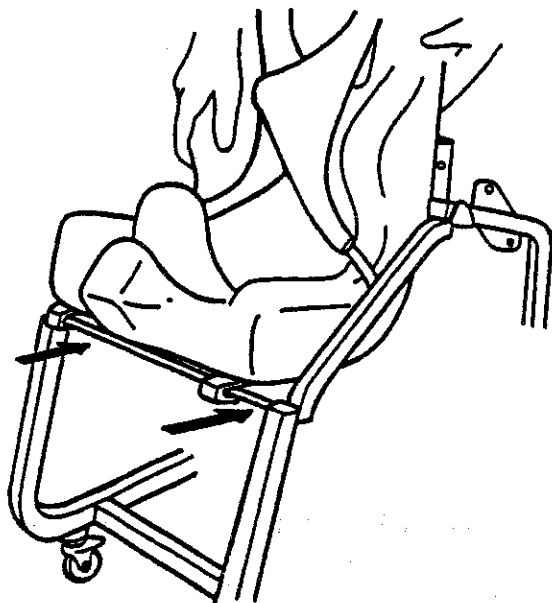


Tighten both plastic thumb screws to secure the seat to the frame.

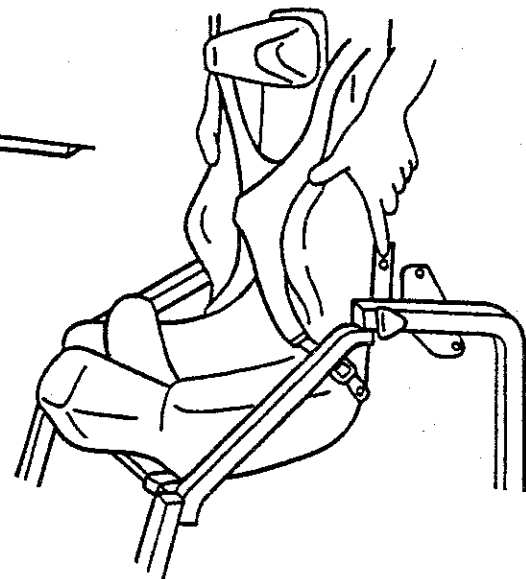

To assemble footrest, hook one side of footrest onto two buttons on side of frame as shown. Swing footrest over, snapping other side onto the two buttons on the opposite side. To adjust reverse assembly process and remove footrest and reassemble moving the footrest to the desired adjustment holes. Make sure that the footrest is positioned over the same adjustment holes on either side of the footrest (I.E. is level).

To use the Footrest: Position the child's foot in the footrest, hooking the back part of the child's shoe. Thread the footrest strap through both "D" rings and then back through the second "D" ring as shown. Pull the strap tight to secure.

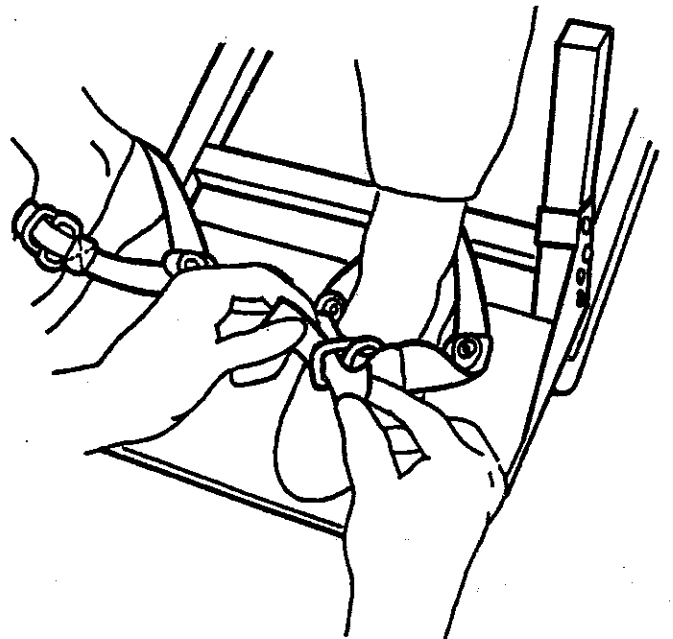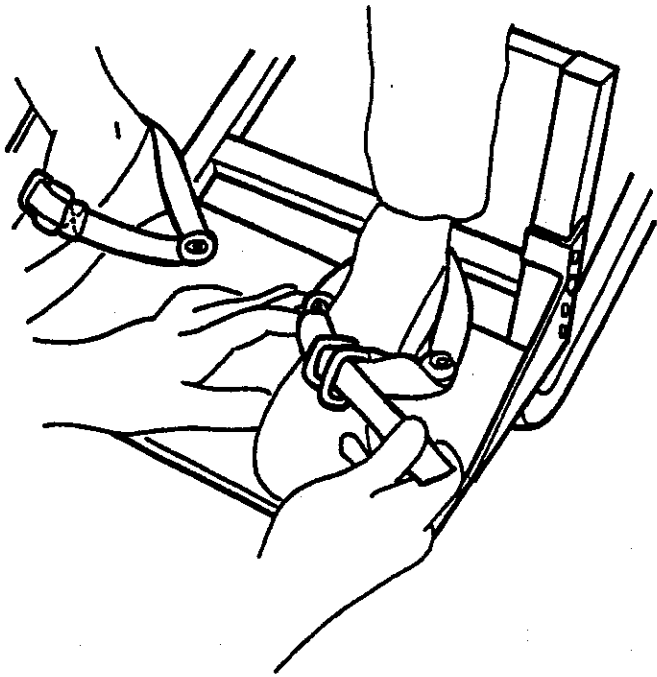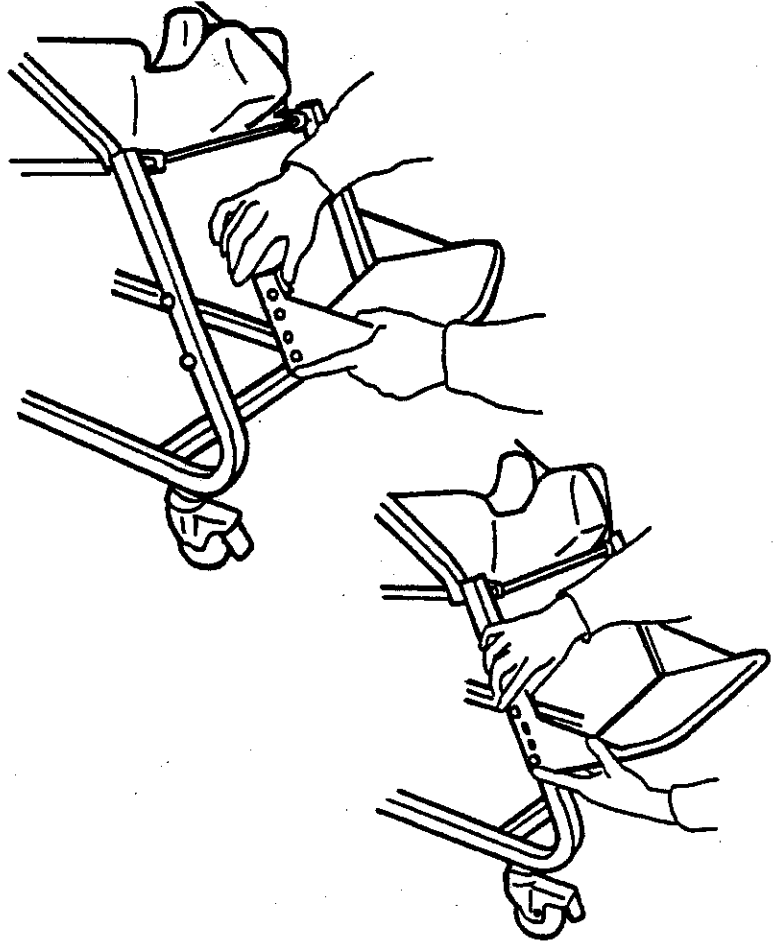



The splash guard is designed to be used with all clients. It protects the Tumble Forms covering from staining and discoloration and helps to direct urine into the pail. It is made of a flexible elastomer so as not to inhibit transfer and aids in providing abduction for the legs.

The splash guard can easily be removed for cleaning by simply pulling it out of the seat from the top. The splash guard should be cleaned and disinfected regularly and can be boiled for sterilization if necessary.



The hole in the seat curves up the back to allow for easy cleanup while child is in seat.

CAUTION: Do not put the splash guard in the microwave to clean, the elastomer will break down if microwaved

The tray can be attached by inserting the tray arms into the arms of the base frame as shown. Once the tray has been positioned appropriately, the black knobs can be tightened (clockwise) to secure tray in place.

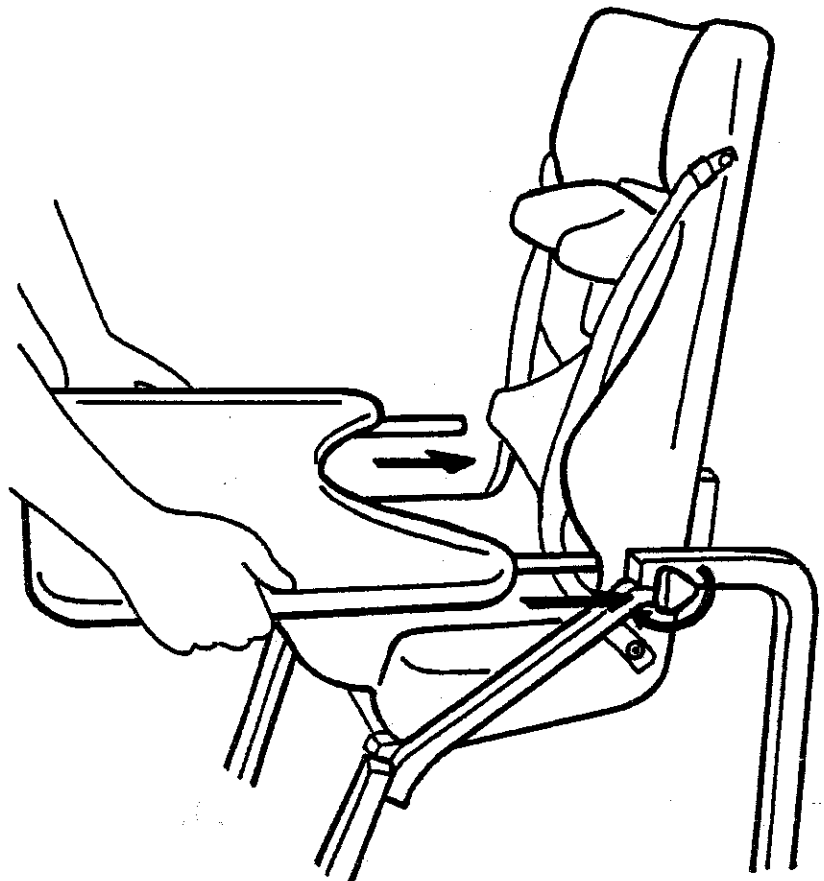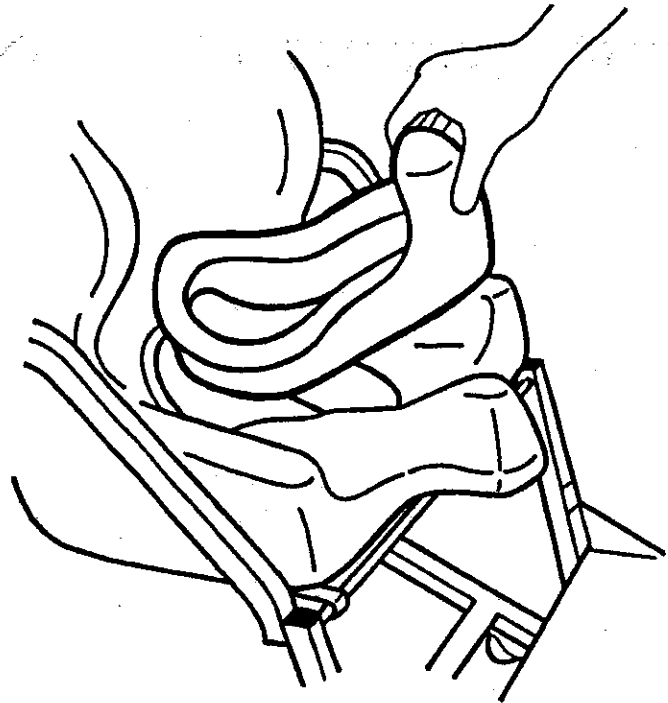

The pail can be inserted and removed by simply sliding it over the pail guides attached to bottom of seat as shown. A disposable plastic bag can be placed in the pail to simplify cleanup. The pail should be cleaned and disinfected regularly.

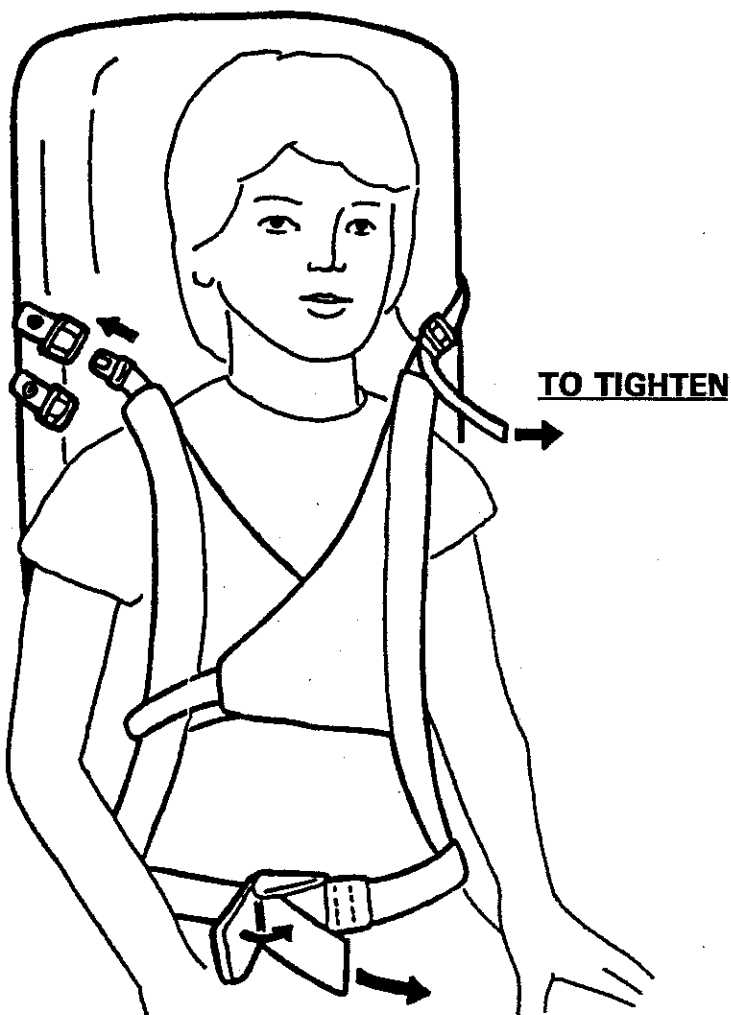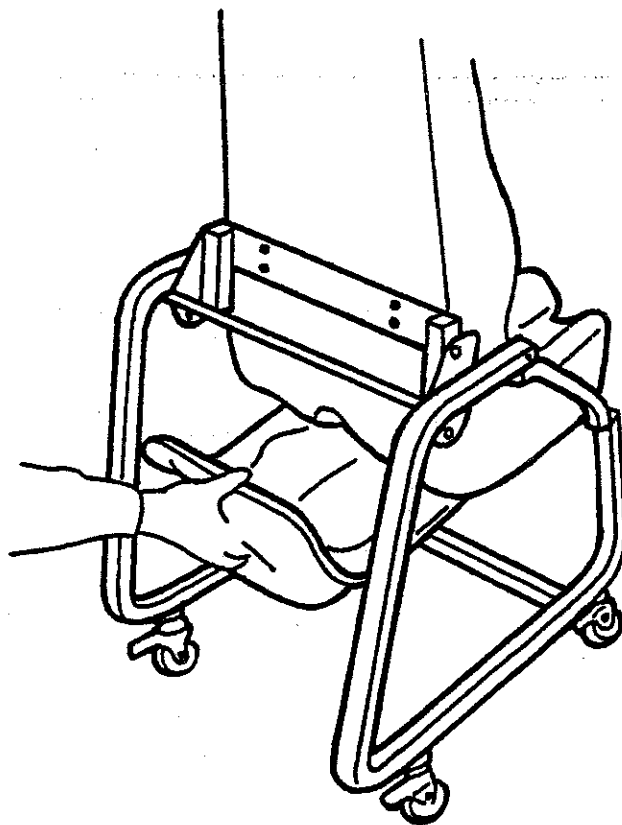


To use: Place child in the Carrie Potty Seat making sure his/her hips are planted firmly back in the seat.

Fasten the hip strap by pulling the strap through the buckle and closing the buckle down over it.



There are two shoulder strap adjustment heights. Adjust accordingly to height of child.

Pull the shoulder straps over the child's arms and remove any slack by tightening the straps at the buckle.

Close the two halves of the chest pad together and wrap the Velcro® strap around to secure. The chest pad should be positioned over the trunk but comfortably below the child's neck line.

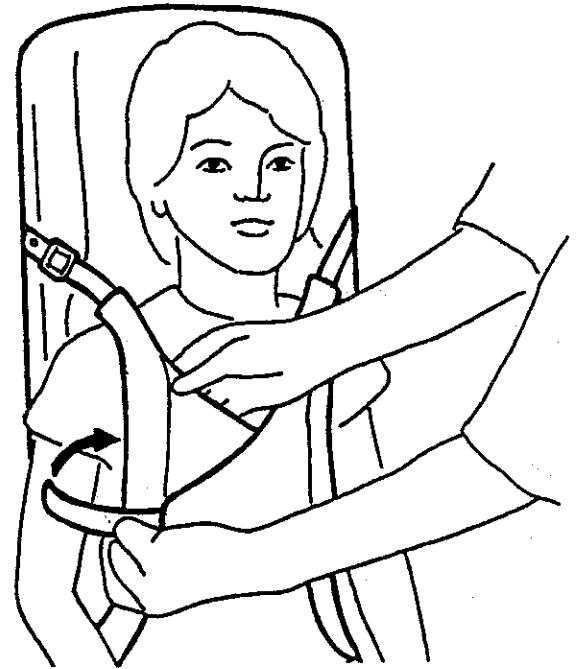

To prevent casters from moving, press the locking tab down as shown.

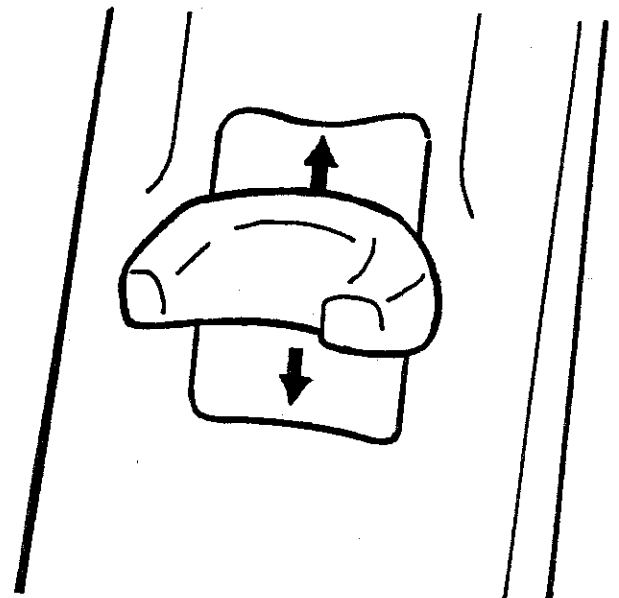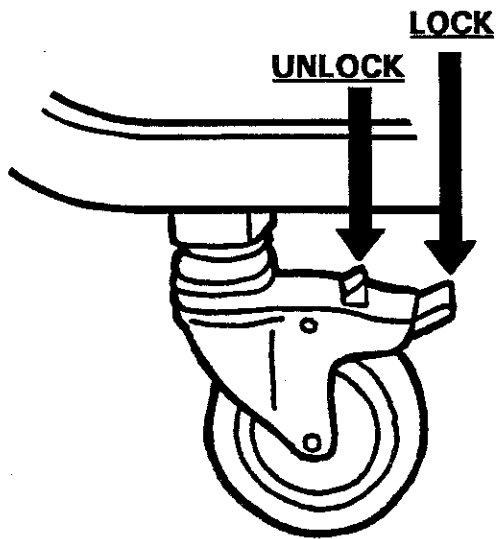

The purpose of the head support is to hold the head and neck in midline. It can be adjusted by attaching it higher or lower on the Velcro® loop pad.

Tumble Forms²[®]

With Anti-Microbial Protection

Please Note: Tumble Forms will not inspect any Carrie Potty Seat returned for warranty claims that has not been thoroughly cleaned and disinfected.

If you have any questions, comments or suggestions please contact:



Sammons Preston Rolyan
4 Sammons Court
Bolingbrook, IL 60440

Call toll free:
1-800-323-5547
In Canada call 1-800-665-9200

Fax toll free:
1-800-547-4333
In Canada fax 1-905-566-9207



Authorized European Representative:
Homecraft, Ltd.
Shelley Close, Lowmoor Road
Industrial Estate, Kirkby-in-Ashfield, Nottinghamshire, NG 17 7ET, UK
Telephone: 44 1623 759146 Facsimile: 44 1623 726886

