

Riser Block Installation

Use the Riser Block included in this package if your child has a leg length discrepancy

Adjust the footrest, as described on page 9 of your Small Adult Carry Seat instruction manual, to accommodate the length of the longer leg.

Remove the footrest bar from the bracket following the instructions on page 9 of your manual.

Remove the two 1/2 inch Philips head screws and two body washers located on the underside of the footrest shoe support that serves the shorter leg.

Follow the drawing to add the riser block.

Use the 1 1/4 inch Allen head screws included in this package to reattach the assembly to the support tube.

